

## Acecide High-Level Disinfectant

An easier, safer, more efficient reprocessing solution for OER-AW.



# Acecide High-Level Disinfectant

**Acecide is a room temperature, peracetic acid-based liquid chemical germicide designed for the Olympus OER-AW—proven for material compatibility by Olympus for Olympus endoscopes.**

## Features and Benefits

1. Unique, easy-to-handle cassette bottles allow for quick and easy chemical loading while minimizing exposure to the disinfectant—enhancing safety for the user.
2. Fast acting PAA delivers high-level disinfection in 5 minutes resulting in a short 18-minute OER-AW cycle time.
3. Time-saving test strip verifies efficacy in just 13 seconds—changing from white to black for clear interpretation of results.
4. Convenient room-temperature (20°C) use protects equipment while saving on utility costs.
5. Only PAA available that has been validated for material compatibility with Olympus endoscopes by the endoscope manufacturer.



## Product Overview

Acecide is an aldehyde-free, reusable liquid chemical germicide designed for exclusive use with the OER-AW. Acecide is delivered in a set. One set of Acecide is loaded into the disinfectant drawer of the OER-AW, when Acecide Solutions 1 and 2 are combined automatically with water to deliver an Acecide Use-Solution for high-level disinfection with PAA as the biocidal agent.

Acecide is reusable when an Acecide test strip reveals that the concentration is above the Minimum Recommended Concentration.

## Product Characteristics

- Minimum Recommended Concentration (MRC): 2,000 ppm (0.2%)
- High-Level Disinfection: 5 minutes at 20°C
- Sporicidal Claim: 10 minutes at 20°C
- No activation required
- Active Ingredient: Peracetic Acid

## Acecide Test Strip

The Acecide Test Strip provides an easy, rapid and convenient means of determining levels of PAA and confirming that Acecide Use-Solution is above the Minimum Recommended Concentration of 2000 ppm. Each bottle contains 100 strips.



*Acecide Test Strip*

Olympus and OER-AW are trademarks or registered trademarks of Olympus Corporation.  
Acecide is a registered trademark of Saraya Co., Ltd.

Specifications, design and accessories are subject to change without any notice or obligation on the part of the manufacturer.

**OLYMPUS**

**OLYMPUS MEDICAL SYSTEMS CORP.**

Shinjuku Monolith, 2-3-1 Nishi-Shinjuku, Shinjuku-ku, Tokyo 163-0914, Japan  
www.olympus-global.com

Global Professional Education website  
www.olympusprofed.com



N8600661-072020