

Acecide High-Level Disinfectant

An easy, safe, efficient reprocessing solution for OER-AW/5.



Acecide High-Level Disinfectant

Acecide is a room temperature, peracetic acid-based liquid chemical germicide designed for the Olympus OER-AW/5—proven for material compatibility by Olympus for Olympus endoscopes.

Features and Benefits

1. Unique, easy-to-handle cassette bottles allow for quick and easy chemical loading while minimizing exposure to the disinfectant— enhancing safety for the user.
2. Fast acting PAA delivers high-level disinfection in 5 minutes resulting in short minutes cycle time (17 minutes for OER-5, 18 minutes for OER-AW).
3. Time-saving test strip verifies efficacy in just 13 seconds—changing from white to black for clear interpretation of results.
4. PAA available that has been validated for material compatibility with Olympus endoscopes by the endoscope manufacturer.



Product Overview

Acecide is an aldehyde-free, reusable liquid chemical germicide designed for exclusive use with the OER-AW/5. Acecide is delivered in a set. One set of Acecide is loaded into the disinfectant drawer of the OER-AW/5, when Acecide Solutions 1 and 2 are combined automatically with water to deliver an Acecide Use-Solution for high-level disinfection with PAA as the biocidal agent.

Acecide is reusable when an Acecide test strip reveals that the concentration is above the Minimum Recommended Concentration.

Product Characteristics

- Minimum Recommended Concentration (MRC): 2,000 ppm (0.2%)
- High-Level Disinfection: 5 minutes at 20°C
- Sporidical Claim: 10 minutes at 20°C
- No activation required
- Active Ingredient: Peracetic Acid

Acecide Test Strip

The Acecide Test Strip provides an easy, rapid and convenient means of determining levels of PAA and confirming that Acecide Use-Solution is above the Minimum Recommended Concentration of 2000 ppm. Each bottle contains 100 strips.



Acecide Test Strip

Olympus and OER-AW/5 are trademarks or registered trademarks of Olympus Corporation. Saraya is manufacturer of Acecide. Acecide is a registered trademark of Saraya Co., Ltd. 2-2-8 Yuzato, Higashiumiyoshi-Ku, Osaka 546-0013, Japan

Specifications, design and accessories are subject to change without any notice or obligation on the part of the manufacturer.

OLYMPUS

OLYMPUS MEDICAL SYSTEMS CORP.

2951 Ishikawa-cho, Hachioji-shi, Tokyo 192-8507, Japan
www.olympus-global.com

Global Professional Education website
www.olympusprofed.com



N8600000-002022